

# GRID MOUNT<sup>®</sup> PAT.

## Pocket Mount Type




### NP-PAN-GR

(For intra-oral film)

## FILM MOUNT with grid scale

for your dental X-ray film filing

Conventionally, grid scale is used for measurement of root canal or for Implant. With GRID MOUNT, fine grid is printed on transparent mount and Image with grid scale is available.

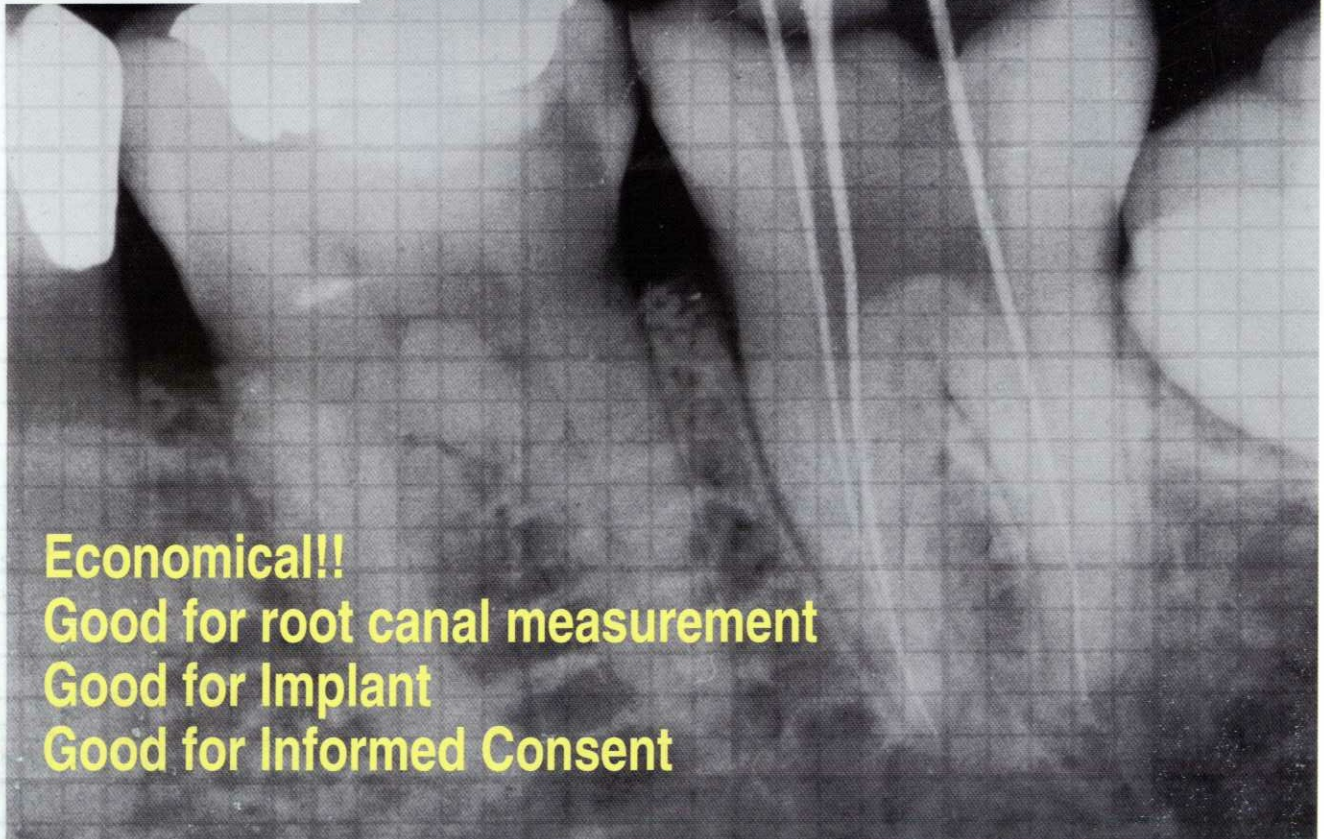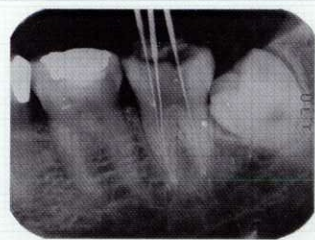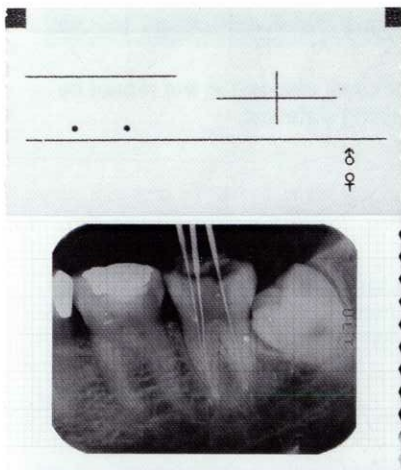

**Economical!!**

**Good for root canal measurement**


**Good for Implant**

**Good for Informed Consent**

### Features of Grid Scale

Measurement is feasible at a glance.

- Fine grid scale lines by special printing technique.
- Grid scale lines printed at 1 mm interval with bold lines at 5 mm interval.
- Grid scale lines do not obstruct visibility of image.

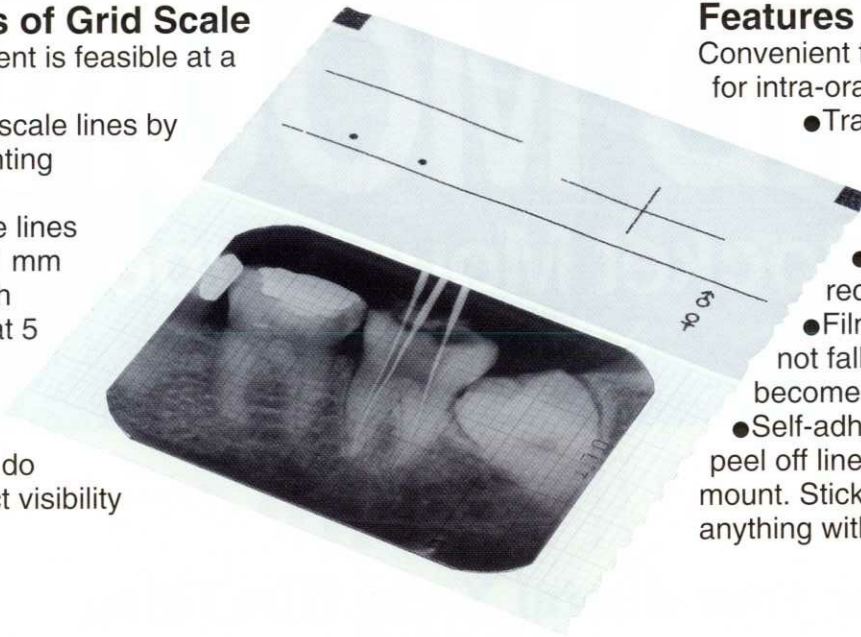


### Features of Mount


Convenient filing and organizing for intra-oral film.

- Transparent mount. Film can be observed in the mount.
- Handy data slip for recording patient data.
- Film in the mount does not fall out as data slip becomes a lid.
- Self-adhesive backing. Just peel off liner on the back of mount. Stick to the chart or anything with flat surface.


### How to use Grid Mount(Pocket Mount type)

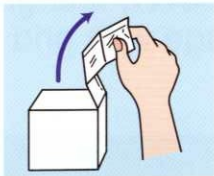


- Grid Mount comes in continuous 480 mounts. Simply pull-out as many as needed.

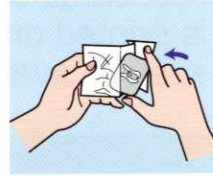


- Film is inserted in the mount by lifting data slip.

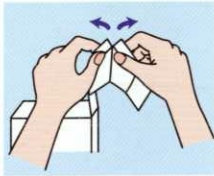


- Tear off at the perforated edge. Gently tear at data slip edge.

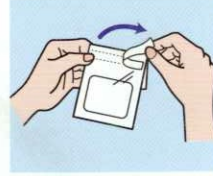

- Just peel off liner on the back of mount and stick to anything with flat surface.

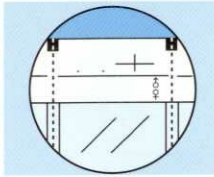


- Handy pre-printed data slip is attached to each mount. Fill in patient data.

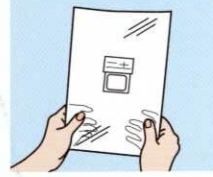

- Can be observed in the mount.

NP-PAN-GR  
480 mounts

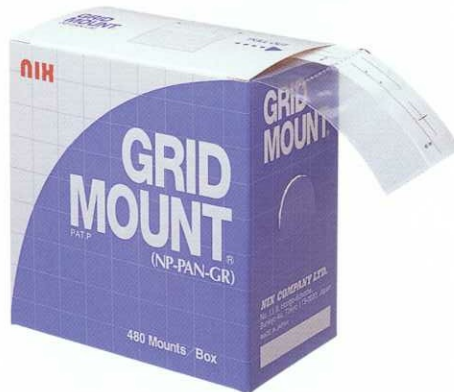

In Europe:



**Hager & Werken GmbH & Co. KG**

Postfach/P.O.B. 10 06 54 D-47006 Duisburg  
\*Ackerstraße 1 D-47269 Duisburg Germany  
Tel: (+49) 202 20 20 0

In U. S. A./Canada:



**Hager Worldwide, Inc.**

13322 Byrd Drive, Odessa, FL 33556, USA  
Tel: (813)926-7474 (800)328-2335  
FAX: (813)926-7473 (800)573-9392

Manufacturer:



**NIX COMPANY LTD.**

Tokyo, Japan